

DivibaX®



The connection device provides a 100% safe mix of powders and fluids between vials and IV-containers




DivibaX® is the result of many years of development and it has a unique and pathbreaking design ensuring optimum user friendliness and safety for the user of the product due to the following:

- It has a unique design with a T-shaped piece being the first horizontal vial application
- It minimises the risk of spreading aerosole toxicological products

- It protects the user against needlestick injuries
- It is 100% protected against run-back of fluid which has always been the great challenge of the market.
- Not possible to separate the components after the mixture has been fulfilled ensuring 100% traceability of the medication thus no double medication.



How to use DivibaX®

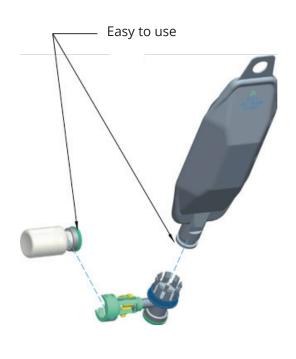



Remove the yellow safety ring and turn the device until it reaches the end of the thread. The syringe penetrates the membrane and vial rubber stopper.

Turn the IV bag and mix the fluids. The mixed solution is now ready to use. The administration set guarantees that no run-back of fluids into the vial is possible. DivibaX® is compatible with all 20 mm ISO standard neck – both for vials and IV containers. The device is sterilised through irradiation.